



APTOS

THREAD LIFTING METHODS

MINIMALLY INVASIVE
THREADS & METHODS

For face and neck
lifting and contouring

MINIMALLY INVASIVE THREADS & METHODS

for face and Neck Lifting

APTOS
THREAD LIFTING METHODS

Aptos is combination of surgical threads and methods of installation. it has been invented by dr. Marlen Sulamanidze in 1996 and developed by doctors: George and Konstantin Sulamanidze. Threads are absorbable and permanent single-use sutures, used for fixation of subdermal tissues in an elevated position. Aptos thread lift is a popular procedure to achieve facial & body refreshment and improve skin quality without having to undergo surgery and the problem of scarring. The procedure is called a minimally invasive with a minimum downtime.

DURING MORE THAN 22 YEARS, WE HAVE DEVELOPED

50 Thread lifting methods

30 Different threads

Minimally invasive procedure

Under local anesthesia

Takes about 20 - 15 minutes

Minimum downtime

Immediately evident results

New collagen growth

Increased microcirculation

Long-term effect

Effective & Proven Results




VISAGE

Excellence Method



Visage Excellence is Universal resorbable surgical thread with multidirectional, specially designed barbs. The unique feature enhances the force of tissue holding. Multidirectional barbs can distribute, stress and hold soft tissues stronger after insertion.

Visage Excellence can be used for fixation of subdermal tissue of the face and neck. Efficient design of **Visage Excellence** approximates the entire treatment area through an easy and non-invasive procedure. Threads are inserted through atraumatic round tip cannula, thus avoiding an extra bleeding and swelling after the procedure. The method is easy and fast to use and needs minimum of downtime for the patient.

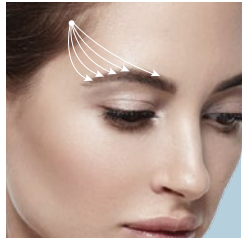


• Brow



• Cheekbone area




• Mental area
• Cheekbone area

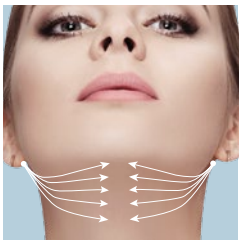

• Submental area
• Neck





EXCELLENCE METHOD

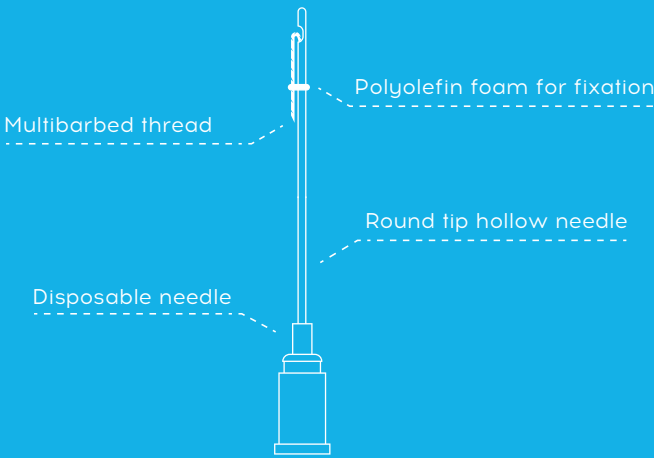

In Pack

- P(LA/CL) thread with barbs USP 2/O, EP 3, 190mm - 10pcs
- Round tip hollow needle 20Gx150mm, straight - 10pcs
- Round tip hollow needle 23Gx80mm, straight - 1pcs
- Lancet point needle 18Gx40mm, straight - 1pcs
- Removable needle attachment
- P(LA/CL) thread with barbs USP 2/O, 120 mm -10 pcs
- Round tip hollow needle 20Gx100mm, straight - 10 pcs
- Lancet point needle 18Gx40mm, straight - 1pcs
- Removable needle attachment
- Infiltration cannula 23Gx80 mm - 1 pcs

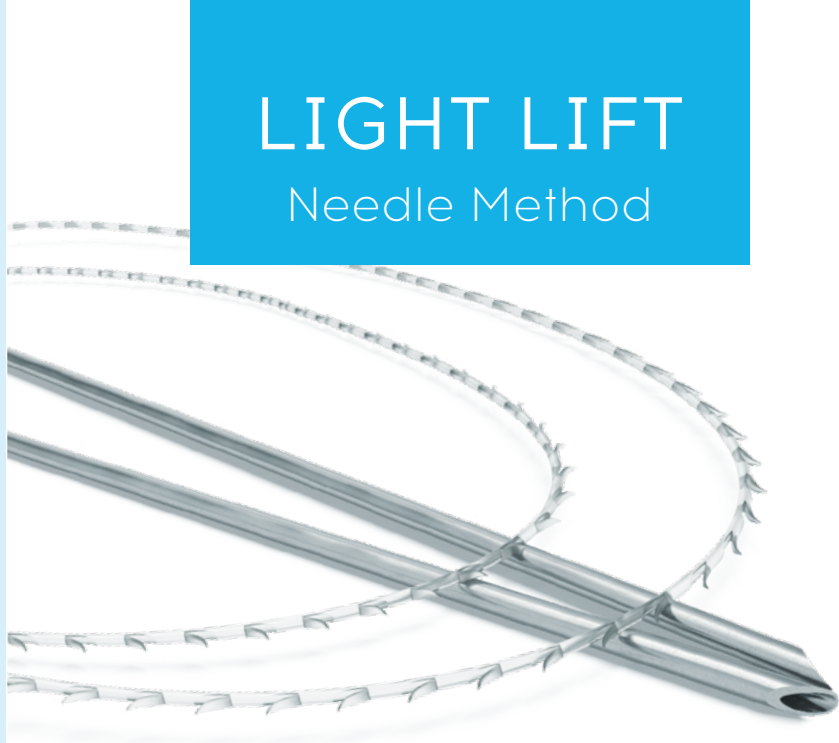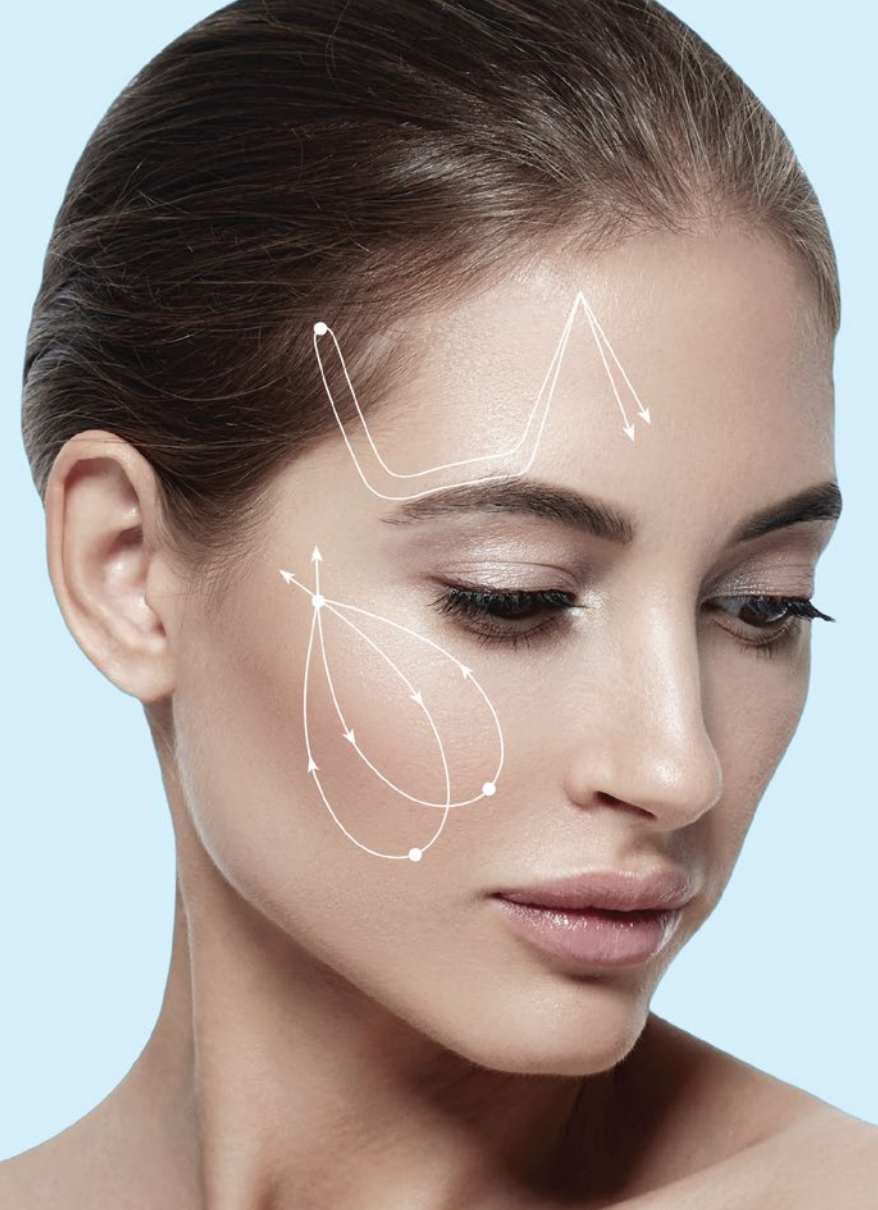


Light Lift Needle is a bidirectional barbed thread which allows minimally invasive and natural approach to the age-related changes of soft tissues. Light Lift Needles are available both in a resorbable and permanent version. They should be inserted into the subcutaneous fat.


Due to the structure, doctors have possibility to correct submental area, eyebrows, with a single suture running from side to side, without taking needles totally out of the skin. This prevents the patient from skin retraction at the exit points along the pathway. New neck shape and angle is immediately apparent. Fixation continues over time.

LIGHT LIFT


Needle Method

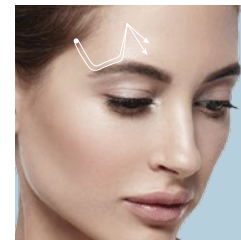

• Brow

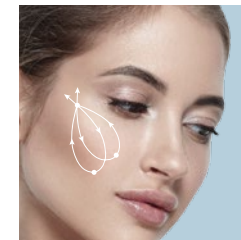

• Cheekbone area

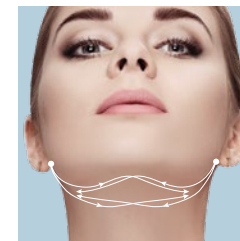

• Submental area



NEEDLE METHOD



LLN2G

- P(LA/CL) thread with barbs USP 2/O, EP3, 500mm-2pcs
- Triangle cutting needle GS1,2x100mm, straight - 4pcs
- Standard needle attachment

LLN2GS

- P(LA/CL) thread with barbs USP 2/O, EP3, 500mm-2pcs
- Double-edged, lancet point hollow needle 1,1x120mm, straight - 4pcs
- Standard needle attachment

AN2G

- Polypropylene thread with barbs USP 2/O, EP3, 500mm-2pcs
- Triangle cutting needle GS1,2x100mm, straight - 4pcs
- Standard needle attachment

AN2GS

- Polypropylene thread with barbs USP 2/O, EP3, 500mm-2pcs
- Double-edged, lancet point hollow needle 1,1x120mm, straight-4pcs
- Standard needle attachment


LLN2GB

- P(LA/CL) thread with barbs USP 2/O, EP 3, 500 mm - 2 pcs
- Double-edged, blunt tip needle 21G x 110 mm - 4 pcs
- Standard Needle attachment

AN2GB

- Polypropylene thread with barbs USP 2/O, EP 3, 500 mm-2 pcs
- Double-edged, blunt tip needle 21G x 110 mm - 4 pcs
- Standard Needle attachment



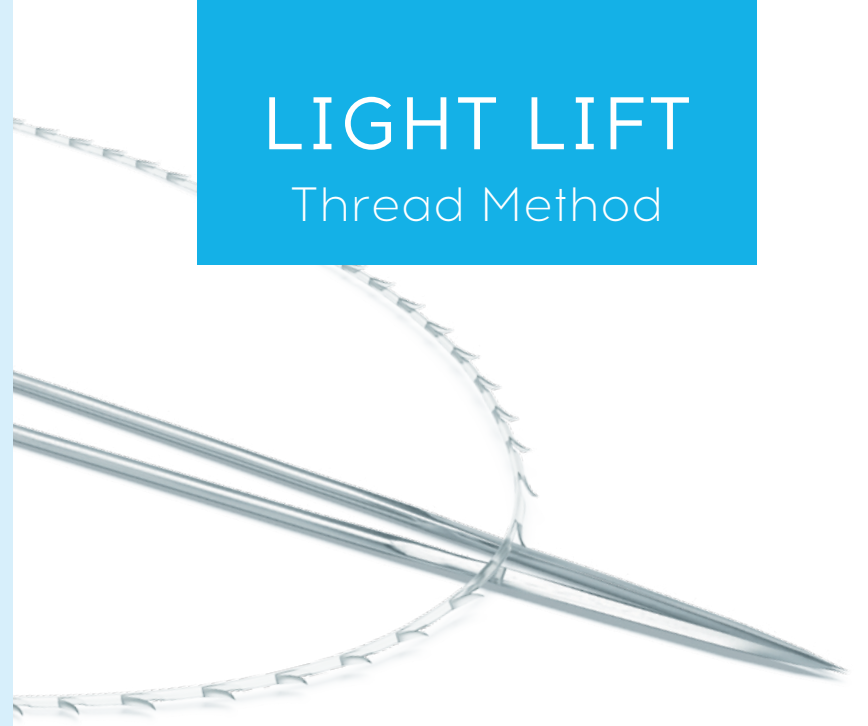

LIGHT LIFT

Thread Method

Light Lift Thread is a bidirectional barbed thread which allows non-invasive and natural approach to the age-related changes of soft tissues. Threads are available both in a resorbable and permanent version. In Light Lift Thread the thread is attached to the end of the straight triangle cutting double needles.

Due to the structure, doctors have possibility to correct eyebrows, jowls, midface and submental area using different pathways that guarantee effective result.

The procedure is easy and fast, with significant reduction of operation time and practically no downtime for the patient.



• Brow




• Cheekbone area



• Cheekbone area



• Mental area

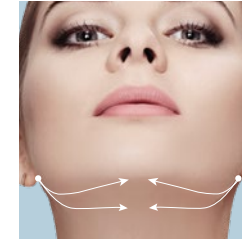

• Submental area



THREAD METHOD



In Pack

LLT2G

- P(LA/CL) thread with barbs USP 2/O, EP3, 250mm-2pcs
- Cutting needle GS1,0x110mm, straight - 4pcs
- Standard needle attachment

LLT2GS

- P(LA/CL) thread with barbs USP 2/O, EP3, 250mm-2pcs
- Lancer point hollow needle 0.83x100mm, straight - 4pcs
- Standard needle attachment

AT2G

- Polypropylene thread with barbs USP 2/O, EP3, 250mm-2pcs
- Triangle cutting needle GS1,0x110mm, straight - 4pcs
- Standard needle attachment

AT2GS

- Polypropylene thread with barbs USP 2/O, EP3, 250mm-2pcs
- Lancer point hollow needle 0.83x100mm, straight - 4pcs
- Standard needle attachment

LLT2GB

- P(LA/CL) thread with barbs USP 2/O, EP 3, 250mm - 2 pcs
- Blunt tip needle 21G x 90mm - 4 pcs
- Standard Needle attachment

AT2GB

- Polypropylene thread with barbs USP 2/O, EP 3, 250mm - 2 pcs
- Blunt tip needle 21G x 90mm - 4 pcs
- Standard Needle attachment






NANO

Excellence Method



Nano Excellence is universal resorbable thread with multidirectional, specially designed barbs for armoring and revitalization. Once inserted in soft tissues, Nano Excellence stimulates new collagen growth, that generates healthy new skin. Nano Excellence can be used for armoring and rejuvenation of various parts of the face, neck and body.

Efficient design of Nano Excellence tightens the skin at the entire treatment area. Threads are inserted through atraumatic blunt tip cannula thus avoiding extra bleeding and swelling after the procedure. The method is easy and fast to use and needs practically no downtime for the patient.

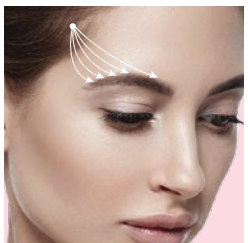


• Brow




• Cheekbone area



• Mental area

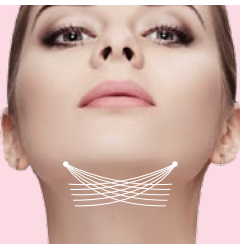


• Submental area

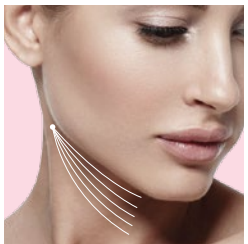

• Neck



EXCELLENCE METHOD

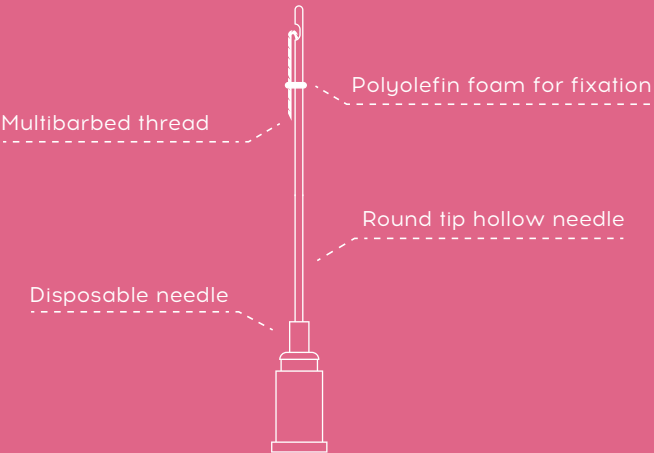

- P(LA/CL) thread with barbs USP 2/0, EP 3, 70mm-5pcs
- Blunt tip needle 22Gx70mm, straight-5pcs
- Removable needle attachment

APTOS

THREAD LIFTING METHODS

Solely Distributed by: _____

DUBAI Medica Trading LLC

Business Bay Al Khaleej Al Thejari
Street, Vision Tower, 38th Floor
P.O.Box: 31135, Dubai - U.A.E.
Tel: + 971 4 43 70 669
Fax: + 971 4 43 70 668
info@medicagroup.com

ABU DHABI Medica Trading LLC

Corniche street, Baynunah
Tower 2, 3rd floor, Office #47
P.O.Box: 43385, Abu Dhabi - U.A.E.
Tel: + 971 2 6581 619
Fax: + 971 2 6581 612
info@medicagroup.com

KSA Medica - RIC

Exit 10, King Abdullah Road,
Sahhab Tower 1, 5th Floor
P.O.Box: 148, Riyadh 11411 - K.S.A.
Tel: + 966 11 46 54 113
Fax: + 966 11 46 30 135
info@medicagroup.com

LEBANON Medica Pharmaceuticals s.a.l.

Jal el Dib, Main Road Mallah Center,
11th Floor
P.O.Box: 90601 Jdeideh - Beirut, Lebanon
Tel: + 961 4 710 973
Fax: + 961 4 710 977
info_me@medicagroup.com

SINGAPORE Medica Singapore Pte Ltd

583 Orchard Road
Suite #14-01, The Forum
P.O.Box: 238884, Singapore
Tel: + 65 67 33 96 89
Fax: + 65 67 33 96 91
info_sg@medicagroup.com